

# SALISBURY AUTO LIFT

## OWNER'S MANUAL



[www.accessunlimitedliftaustralia.com.au](http://www.accessunlimitedliftaustralia.com.au)  
1/44 CHEVIOT ROAD  
SALISBURY SOUTH, SOUTH AUSTRALIA 5106  
PH. 08 8182 3510, F. 08 8182 3509, e. [farmeast@senet.com.au](mailto:farmeast@senet.com.au)

# SAFETY GUIDELINES



## **WARNING!**

Please follow all of the instructions in this manual.

The symbols below are used throughout this owner's manual and on the product to identify warnings and important information.

It is very important for you to read them and understand them completely.



## **WARNING!**

Indicates a potentially hazardous condition/situation. Failure to follow designated procedures can cause either personal injury, component damage, or malfunction. On the product, this icon is represented as a black symbol on a yellow triangle with a black border.



## **MANDATORY!**

These actions should be performed as specified. Failure to perform mandatory actions can cause personal injury and/or equipment damage.



## **PROHIBITED!**

These actions are prohibited. These actions should not be performed at any time or in any circumstances. Performing a prohibited action can cause personal injury and/or equipment damage. On the product, this icon is represented as a black symbol with a red circle and red slash.

**Please fill out the following information for quick reference:**

Purchase Date:	
Warranty expire:	
Serial Number:	

## **NOTE:**

This owner's manual is compiled from the latest specifications and product information available at the time of publication. We reserve the right to make changes as they become necessary. Any changes to our products may cause slight variations between the illustrations and explanations in this manual and the product you have purchased. The latest/current version of this manual is available on our website.

## **Distributed by:**

### **Access Unlimited Lift Australia**

1/44 CHEVIOT ROAD

SALISBURY SOUTH, SOUTH AUSTRALIA 5106

PH. 08 8182 3510, F. 08 8182 3509, e. [farneast@senet.com.au](mailto:farneast@senet.com.au)

# INTRODUCTIONS

Thank you for purchasing Salisbury Auto Lift. Your safety is very important to us. Please read and follow all instructions in this manual before operating your lift system for the first time. Your understanding of these instructions is essential for the safe operation of your new lift system.

We are not liable for damage to property or personal injury arising out of the unsafe use of a lift system. We are also not liable for any property damage or personal injury arising out of the failure of any person and/or user to follow the instructions and recommendations set forth in this manual.

## **PURCHASER'S AGREEMENT**

By accepting delivery of this product, you promise that you will not change, alter, or modify this product or remove or render inoperable or unsafe any guards, shields, or other safety features of this product; fail, refuse, or neglect to install any retrofit kits from time to time provided to enhance or preserve the safe use of this product.

# SAFETY

## WEIGHT LIMITATIONS

The lift system is designed to lift a maximum user weight of 135 kg.

## MANDATORY



Stay within the specified weight capacity of the lift system. Exceeding the weight capacity voids your warranty. Pride will not be held responsible for injuries and/or property damage resulting from failure to observe weight limitations.

## PRE-LIFT INSPECTION

There should be no sharp edges on Salisbury Auto Lift and the fitting kit. If you find any sharp edges, please contact us before use.

Misuse of the product could result in injury.

Inspect the sling and straps before every use for twisting, fraying or signs of wear. If signs of wear become evident, have the sling replaced. Check that the positioning of the person being lifted is correct at the start of the lift. Refer to the technical section for the correct method of threading the straps through the hooks to avoid slipping.



### WARNING

A frayed or worn strap can snap, resulting in injury to the person being lifted and/or the operator. Do not use a damaged sling.

## OPERATOR POSITIONING

Keep hands clear of the lift arms during motion. Salisbury Auto Lift is only doing the lifting, control the lateral movement using the handle on the back of the sling. Stand close to the person being lifted to be able to safely control their movement, refer to the operating instructions for detailed guidance.

## SAFETY POINTS



### **WARNING**

The purpose of the Salisbury Auto Lift is to transfer a person simply, safely and with a degree of comfort, from a mobility product to a vehicle or other seat and vice versa. It must not be used for any other purpose.



### **WARNING**

Ensure all straps are set correctly. All metal clips are positively located on their lugs before lifting. Do not lift/lower at the same time as transferring the person into and out of the vehicle.



### **WARNING**

Keep hands, clothing and electrical cord of hand control clear of all moving parts during lifting/lowering operations. The handset can be placed on top of the lift arm when not in use. Do not use the handset whilst it is attached to the body of the lift.



### **WARNING**

Avoid catching the handset cord in the car door or moving parts, resulting in damage to the cord and loss of hoist function



### **WARNING**

During wet weather avoid getting the electrical connection and vicinity wet.



### **WARNING**

Never use a system that has signs of damage.



### **WARNING**

Do not exceed maximum lifting capacity.

## PRODUCT SPECIFICATIONS

- Lifter weight in total 10 kg, easily taken apart into 4 pieces
- Battery weight 7 kg.
- Lifting capacity 135 kg
- Maximum height range of lift: 62.2 cm
- Fuse rating: 25 Amp
- Power supply: Gel battery 12V
- Slings available: Extra small - Extra large including amputee sling


## TECHNICAL INFORMATION

### SLING



The lifting straps of the sling should be adjusted so that each strap is as short as possible, giving the best lift and the most manoeuvrability for the person

### SPEEDY BAR

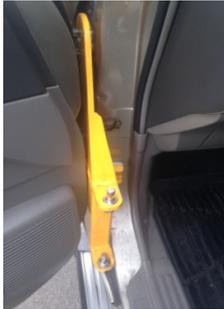

Speedy bar is the part that will hold the main lifter. Insert on to door hinge and make sure it sits properly. Speedy bar should be removed from the door hinge before you shut the door.

### MAIN LIFTER



Main body lifter is the part with the engine contains extension arm and lift arm.

### BATTERY CONNECTION



This lifter power is from battery gel supplied. You will have to supply the charger. Contact us if you need information.

### REMOTE CONTROL




Button up and down. Releasing button will stop from lowering or lifting

# OPERATIONS

## ENTERING THE VEHICLE

1. Insert speedy bar into door hinge

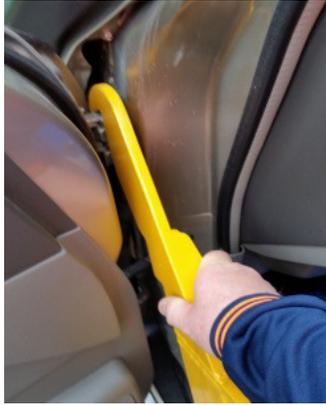

2. Attach main body lifter onto speedy bar



3. Connect main body lifter to the battery



4. Place the sling under the person to be lifted



5. Hook the sling into lift arm, front hook first



6. Hook both side of the sling into lift arm



7. Press the up button to lift



8. Once the person slightly higher than car seat, remove the wheel chair



9. Push the person into the car, legs first and seated. Mind the head when pushing into the car



10. Remove lifter, battery and speedy bar. Leave sling under the passenger ready for unloading

11. Fasten seatbelt, shut the door and ready to go

## **EXITING THE VEHICLE**

Exiting the vehicle is the vice versa of the entering the vehicle

## CARE & MAINTENANCE

1. Inspect sling for tear. Sling can be washed in the washing machine.
2. Inspect cord for damage sign
3. Inspect pivot point. You may lubricate the pivot point every six month to smoother movement.
4. Main body lifter can be cleaned with damp cloth.

**Do not immerse in water, do not allow water runs into electrical connection. Do not load or unload passenger when it's raining except you're loading and unloading undercover.**

## WARRANTY

The warranty period for the Salisbury Auto Lift is 12 months from the date of delivery.

This warranty **will cover** any **defects of workmanship or function**.

The warranty **does not cover** any incorrectly installation or one with damaged mechanical or electrical components due to **incorrect use or damage**.

No alteration should be made.